


### 6.6.2. Cleaning

#### 1. Cleaning objective lens and anterior segment adopter

- ① Cover the objective lens with lens cap to protect the lens from external pollution.
- ② Use blower for removing dust on the surface of lens.

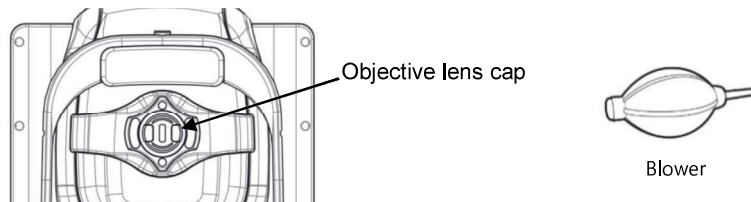

- ③ To remove smear on the surface of lens, use lens cleaning paper or cotton swab moistened with alcohol.
- ④ Do not use a metal or hard stick. Also do not use rubbing alcohol because coating of lens may be damaged or removed.
- ⑤ If Anterior segment adapter is used for patients with any infectious disease, be sure to clean the Anterior segment adapter with cotton swab moistened with alcohol to prevent secondary infection.

#### 2. System exterior

- ① Keep system exterior clean with soft cloth. For severe stains, wipe with a soft cloth with neutral detergent diluted with water. Do not use organic solutions such as thinner or benzene.
- ② Wipe the touch screen with dry soft cloth. Do not use sponge or cloth soaked with large amount of liquid.
- ③ Do not press hard or place magnetic objects near the touch screen.

#### 3. Part of patient contact

- ① Wipe the headrest and the chinrest with a clean cotton swab or gauze. For severe stains, use a soft cloth with alcohol.
- ② Remove a single sheet of chinrest paper if the chinrest paper is used.

#### 4. Others

- ① Cover device with dustcover for unused storage for a long time.
- ② Clean headrest and chinrest with alcohol before sending device to authorized agent or Huvitz for maintenance.
- ③ The rubber ring inserted to conceal the wires may be out during use. It can be used either by inserting it again or by removing it.

